

Molex Knight™

一次性防护口罩

Disposable Protective Mask

产品说明


Products Information



福建苏卡食品有限公司
FUJIAN SUKA FOOD CO.,Ltd.

目录 Contents

资质证书 Certifications.....	1
产品说明 Products Information.....	2
使用说明 Instructions.....	3
使用方法 Usage.....	4
包装样式 Packaging.....	5

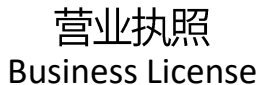

产品说明 Certifications

【产品名称】 一次性日常防护型口罩

【型号规格】 大号（普通成人款） 【颜色】 蓝色

【材质】 本品内外层为纺粘无纺布，中间层为熔喷无纺布。塑料鼻梁条加强密封性，呢绒弹力耳挂。

【保质期】 两年

【注意事项】 本产品限一次性使用，用后销毁。内包装破损，过期产品禁止使用。

对产品材料过敏者禁用。

[Product Name] Disposable Protective Mask

[Size] One Size Fit Most (Adult) **[Color]** Blue


[Materials] 3 layer structure. The inner and outer layer is spunbonded nonwoven, and the middle layer is melt blown nonwoven. Plastic nose b to enhance the sealing, Woollen elastic earloop.

[Shelf Life] 2 years

[Notice] One-time use product, destroyed after use.

Forbidden to use if the inner packing is damaged, and products expired.

Use sparingly for those with allergies.

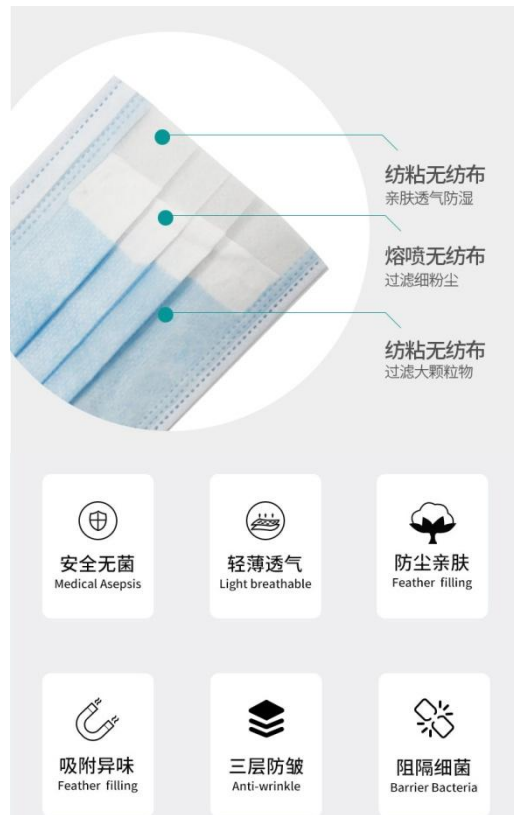

使用说明 Instructions

产品使用说明：

- 1.确认在有效期内和包装完好后，撕开包装，取出口罩；
- 2.将罩体遮盖口、鼻，松紧带挂在耳朵上或将系带系在耳朵上；
- 3.按捺鼻夹，使之与鼻梁吻合；
- 4.使用完毕，集中销毁。

Product Instruction

- 1.Make sure the package is complete, and the mask within the expiry date. Tear off the package, and take out the mask.
- 2.Cover the mouth and nose with the mask, hang the elastic band over the ears.
- 3.Press down the nose clip to match the nose.
- 4.Destroyed after use.

使用方法 Usage

产品使用方法:

- ①正面朝外，双手各拉一边耳带；
- ②口罩先抵住下巴，将耳带拉向耳后挂在耳朵上，调整至感觉舒适；
- ③将双手手指置于鼻梁位置，检查密合性，调整至舒适；
- ④使用双手贴面，检查与脸部的密合性
- ⑤推荐使用时间：4小时

Directions for use:

- 1.Blue color is the outward side, pull the earloop with both hands;
- 2.Mask against the chin, pull the earloop behind the ear, adjust to comfortable position;
- 3.Fingers on the nose, check the tightness, adjust to feel comfortable;
- 4.Apply the face with both hands and check for facial tightness.
- 5.Recommended duration: 4 hours



包装样式 Packaging



独立包装 一盒50片
Individually packed.50 pcsa box.

产品规格: 17cm X 9.5cm
Product Size: 17cm x 9.5cm

盒子规格: 21 x 10 x 10cm
Box Size: 21 x 10 x 10cm

一箱40盒
40 boxesa Pack

装箱规格: 52.5 x 44 x 42.5cm
Packing Size: 52.5 x 44 x 42.5cm

装箱毛重: 9.07kg
Gross Weight: 9.07kg

装箱净重: 6.28kg
Net Weight: 6.28kg

檢 測 報 告				No. FZ000130	
檢 測 單 據				第 三 頁 共 五 頁	
<div> <div>  </div> <div> 報告編號: 1700001 檢驗項目: 衛生用品類 檢驗日期: 2017-01-05 10:00:00 </div> </div>					
檢驗項目	標準/標準編號	判定標準	判定結果	備註	備註
1. 外觀質量	GB 19840-2005	100%	合格		
2. 性能質量	GB 19840-2005	100%	合格		
3. 化學性能質量	GB 19840-2005	100%	合格		
4. 物理性能質量	GB 19840-2005	100%	合格		
5. 衛生性能質量	GB 19840-2005	100%	合格		
6. 安全性能質量	GB 19840-2005	100%	合格		
7. 衛生性能質量	GB 19840-2005	100%	合格		
8. 安全性能質量	GB 19840-2005	100%	合格		
9. 衛生性能質量	GB 19840-2005	100%	合格		
10. 安全性能質量	GB 19840-2005	100%	合格		
11. 衛生性能質量	GB 19840-2005	100%	合格		
12. 安全性能質量	GB 19840-2005	100%	合格		
13. 衛生性能質量	GB 19840-2005	100%	合格		
14. 安全性能質量	GB 19840-2005	100%	合格		
15. 衛生性能質量	GB 19840-2005	100%	合格		
16. 安全性能質量	GB 19840-2005	100%	合格		
17. 衛生性能質量	GB 19840-2005	100%	合格		
18. 安全性能質量	GB 19840-2005	100%	合格		
19. 衛生性能質量	GB 19840-2005	100%	合格		
20. 安全性能質量	GB 19840-2005	100%	合格		
21. 衛生性能質量	GB 19840-2005	100%	合格		
22. 安全性能質量	GB 19840-2005	100%	合格		
23. 衛生性能質量	GB 19840-2005	100%	合格		
24. 安全性能質量	GB 19840-2005	100%	合格		
25. 衛生性能質量	GB 19840-2005	100%	合格		
26. 安全性能質量	GB 19840-2005	100%	合格		
27. 衛生性能質量	GB 19840-2005	100%	合格		
28. 安全性能質量	GB 19840-2005	100%	合格		
29. 衛生性能質量	GB 19840-2005	100%	合格		
30. 安全性能質量	GB 19840-2005	100%	合格		
31. 衛生性能質量	GB 19840-2005	100%	合格		
32. 安全性能質量	GB 19840-2005	100%	合格		
33. 衛生性能質量	GB 19840-2005	100%	合格		
34. 安全性能質量	GB 19840-2005	100%	合格		
35. 衛生性能質量	GB 19840-2005	100%	合格		
36. 安全性能質量	GB 19840-2005	100%	合格		
37. 衛生性能質量	GB 19840-2005	100%	合格		
38. 安全性能質量	GB 19840-2005	100%	合格		
39. 衛生性能質量	GB 19840-2005	100%	合格		
40. 安全性能質量	GB 19840-2005	100%	合格		
41. 衛生性能質量	GB 19840-2005	100%	合格		
42. 安全性能質量	GB 19840-2005	100%	合格		
43. 衛生性能質量	GB 19840-2005	100%	合格		
44. 安全性能質量	GB 19840-2005	100%	合格		
45. 衛生性能質量	GB 19840-2005	100%	合格		
46. 安全性能質量	GB 19840-2005	100%	合格		
47. 衛生性能質量	GB 19840-2005	100%	合格		
48. 安全性能質量	GB 19840-2005	100%	合格		
49. 衛生性能質量	GB 19840-2005	100%	合格		
50. 安全性能質量	GB 19840-2005	100%	合格		
51. 衛生性能質量	GB 19840-2005	100%	合格		
52. 安全性能質量	GB 19840-2005	100%	合格		
53. 衛生性能質量	GB 19840-2005	100%	合格		
54. 安全性能質量	GB 19840-2005	100%	合格		
55. 衛生性能質量	GB 19840-2005	100%	合格		
56. 安全性能質量	GB 19840-2005	100%	合格		
57. 衛生性能質量	GB 19840-2005	100%	合格		
58. 安全性能質量	GB 19840-2005	100%	合格		
59. 衛生性能質量	GB 19840-2005	100%	合格		
60. 安全性能質量	GB 19840-2005	100%	合格		
61. 衛生性能質量	GB 19840-2005	100%	合格		
62. 安全性能質量	GB 19840-2005	100%	合格		
63. 衛生性能質量	GB 19840-2005	100%	合格		
64. 安全性能質量	GB 19840-2005	100%	合格		
65. 衛生性能質量	GB 19840-2005	100			

[illegible]

[illegible]



公司厂房



净化生产车间

日产量可达百万个口罩

执行标准：GB/T32610-2016

生产企业名称：福建苏卡食品有限公司生产

地址：福建省泉州市晋江市五里经济开发区（五里园）力马路29号

电话：0595-85085692